



The new Agger's Rumen Infusion Pump

- Robust metal pump and shaft for long term durability.
- Ergonomic handle for ease of use.
- Only 1kg and conveniently sized, so still easy to transport and handle.
- Simple, durable one way valve which pumps 20 litres in only 60 seconds.
- Connects easily to the Agger's Drenching Kit via the snap fit connector.
- Tried and tested by some of the busiest UK cattle vets, with excellent feedback.
- Available from your wholesaler now.



For use with the Agger's Drenching Kit



Care instructions

Before and after each use, wash through the Agger's pump and the Agger's drench set with hot soapy water (to a maximum of 50°C) and rinse.

Drenching instructions

To insert:

1. Ensure the tip of the hose (the bolus) is completely drawn back against the mouth guard with no hose showing.
2. Hold the corner of the cow's mouth with your free hand and insert the bolus and the mouth guard over the tongue until the nose clip can be secured in the nostrils.
3. Turn the head to the right and push the plastic hose through the mouth guard until approx. 1 to 1.5m of hose is left. Whilst you are doing this, check that the hose is going down the oesophagus by placing your hand on the left side of the neck; you should be able to feel the hose as it travels down the oesophagus.
4. The bolus should now be in the rumen. If you are not sure smell the 'hose to pump' connector end of the hose and you will be able to detect the odour of rumen fluids.
5. Fix the hose by twisting the locking mechanism to lock the hose in place.
6. Pump slowly with even strokes.
7. Seek professional advice before using. If you are in any doubt that the bolus is not in the rumen, do not use the pump.

To remove:

1. Twist back the locking mechanism to release the hose lock.
2. Pull the hose back removing it from the rumen. Be sure to pull it all the way back until the bolus is against the mouth guard.
3. Remove the nose clip and pull out the mouth guard.

Part of the Agger's range from TVM UK

Suggested use	Agger's Glycerol Plus	Agger's Pro Calcium	Agger's Pro Liver	Agger's Fresh Cow	Agger's Restore	Agger's Off Feed	Agger's Antacid
An aid for freshly calved cows at risk of milk fever, ketosis and displaced abomasums	✓			✓			
An aid for cows at risk of dehydration caused by failure to drink, diarrhoea, surgery or peritonitis.					✓		
An aid for cows at risk of ketosis	✓					✓	
Inappetance in early lactation or anorexic cows - aid to restoring hydration, rumen pH and providing energy							✓
Acute rumen acidosis and heat stress							
An aid for risks of a fatty liver	✓		✓				
An aid for the reduction of milk fever		✓		✓			

 TVM UK Animal Health Ltd
 Kirtlington Business Centre
 Kirtlington, Oxon OX5 3JA
 United Kingdom

 0800 038 5868

 help@tvm-uk.com


tvm UK
 INNOVATIVE
 ANIMAL HEALTH